# Masimo SafetyNet Alert™

User's Manual

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## For Export Only, Not for Sale in the USA

There may be information provided in this manual that is not relevant for your system. Do not operate Masimo SafetyNet Alert without completely reading and understanding these instructions. If you encounter any serious incident with product, please notify the competent authority in your country and the manufacturer.

Wireless Radio:

Contains FCC ID: 2AC7Z-ESP32SOLO1; FCC ID: VKF-CONNHUB

Contains IC: 21098-ESP32SOLO1; IC: 7362A-CONNHUB

Masimo Corporation  
52 Discovery  
Irvine, CA 92618, USA  
Tel.: 949-297-7000  
Fax.: 949-297-7001  
[www.masimo.com](http://www.masimo.com)

EU authorized representative for Masimo Corporation:

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Patents: [www.masimo.com/patents.htm](http://www.masimo.com/patents.htm)

Masimo® and ® are federally registered trademarks of Masimo Corporation.

Masimo SafetyNet™ and Masimo SafetyNet Alert™ are trademarks of Masimo Corporation.

All other trademarks and registered trademarks are property of their respective owners.

© 2021 Masimo Corporation

# Contents

---

About This Manual.....	5
Getting Started Guides.....	5
In-App Videos.....	5
Product Description and Intended Use.....	5
Product Description.....	5
Who the Device is intended to be used on (Indications for Use).....	6
Safety Information.....	6
Safety Warnings.....	6
Performance Warnings.....	7
Cleaning and Service Warnings.....	9
Compliance Warnings.....	10
Risks and Benefits.....	12
Your Body and Oxygen.....	12
Risks of Masimo SafetyNet Alert.....	12
Benefits of Masimo SafetyNet Alert.....	13
Description.....	15
Masimo SafetyNet Alert System.....	15
Features.....	16
Basic Setup and Use.....	19
Getting Started.....	19
Step 1: Smart Phone Setup.....	19
Step 2: Install the Masimo SafetyNet Alert App.....	20
Step 3: Masimo SafetyNet Alert System Setup.....	20

Step 4: View Live Data .....	28
Advanced Use .....	35
History .....	35
Troubleshooting.....	41
Masimo SafetyNet Alert Messages .....	41
Troubleshooting Masimo SafetyNet Alert.....	47
Troubleshooting Measurements.....	52
Appendix.....	53
Specifications.....	53
Service and Maintenance.....	68

# About This Manual

---

## Getting Started Guides

Use the quick reference guide included for the following:

- Basic Setup and starting your first monitoring session.

## In-App Videos

Watch the videos in your app to find out more about:

- Setting up your Masimo SafetyNet Alert System.
- Masimo Sensor placement.
- Connecting your Masimo SafetyNet Alert System to WiFi.

## Product Description and Intended Use

---

### Product Description

**Masimo SafetyNet Alert** - a system that provides spot-checking and continuous monitoring of physiological data by wireless communication to medical technologies. The system includes the following pieces:

**Masimo SafetyNet Alert App** - software application installed on a smart phone that provides the graphical user interface to display your data and alarm condition status.

**Masimo Sensor** - wireless, wearable sensor that provides the physiological data.

**Masimo Chip** - attaches to the wireless Masimo Sensor and connects wirelessly to the Bedside Station.

**Bedside Station** - device that communicates monitoring data wirelessly from the medical technologies to the Masimo Cloud.

**Masimo Cloud** - a server accessed over the internet that gathers and stores measured data communicated wirelessly from the Bedside Station.

# Who the Device is intended to be used on (Indications for Use)

## Intended Use

The Masimo SafetyNet Alert is intended for the spot-checking or continuous monitoring of functional oxygen saturation (SpO<sub>2</sub>) and Pulse Rate (PR). The subject device provides the management of notifications and the ability to display and transfer aggregated data communicated from compatible medical technologies.

## Indications for Use

The Masimo SafetyNet Alert is indicated for the spot-checking and continuous monitoring of functional oxygen saturation of arterial hemoglobin (SpO<sub>2</sub>) and Pulse Rate (PR) in adult, pediatric, infants, and neonates during motion, no-motion and low perfusion conditions in hospital and home environments.

## Safety Information

---

Before using the Masimo SafetyNet Alert, read the following safety information carefully.

### Safety Warnings

**WARNING:** For the safe use, do not use if it appears damaged.

**WARNING:** For your safety, do not try to repair a device that is damaged.

**WARNING:** Place the Bedside Station where it will not fall on anyone.

**WARNING:** Make sure the oxygen level is displayed to ensure that monitoring has started.

**WARNING:** Do not place the Bedside Station in areas where there is a lot of flammable gases such as anesthetics, oxygen, or nitrous oxide are present to prevent risk of fire.

**WARNING:** During defibrillation, avoid contact with any devices applied to the person to prevent the electric shock being transferred.

**WARNING:** For safe use avoid the following:

- Do not place the Bedside Station on a wet surface.

- Do not soak any part of the system in liquid.
- Do not try to sterilize.
- Only use Masimo recommended solutions for cleaning your system. See ***Cleaning*** on page 68.
- Do not clean while the device is in use.

**WARNING:** Keep small parts away from small children. Small items can be a choking hazard.

**WARNING:** Carefully position any cables to avoid possible strangulation or entanglement.

**CAUTION:** Place the Bedside Station where you can easily disconnect it from AC power in case of an emergency.

**CAUTION:** Only use the AC power supply included with your Bedside Station to prevent damage to the device.

**Note:** Do not monitor more than one person at a time with Masimo SafetyNet Alert.

## Masimo Sensor

**WARNING:** Avoid wrapping the sensor too tightly around your finger to avoid injury.

**WARNING:** To avoid skin injury, consider moving the sensor to another finger after it has used more than 8 hrs.

## Performance Warnings

**WARNING:** Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your physician.

**WARNING:** Do not use Masimo SafetyNet Alert as an apnea monitor. Masimo SafetyNet Alert is designed to monitor oxygen levels and not apnea.

**WARNING:** Only use Masimo-approved parts with Masimo SafetyNet Alert to make sure the device works correctly.

**WARNING:** Check your device is working before use for longer periods by checking to see your starting oxygen level.

**WARNING:** Avoid placing liquids on or near the Bedside Station. Liquids spilled on Masimo SafetyNet Alert may affect how it works.

**WARNING:** Do not place anything on top of the Bedside Station to avoid damaging or blocking the wireless signal or muffling the audible alarms.

**WARNING:** The following factors can cause false oxygen measurements:

- Sensor is applied the incorrectly
- Restricted blood flow to the sensor site
- A sensor site with nail polish, acrylic nails, glitter, etc.
- A sensor site with moisture, birthmarks, skin discoloration, or foreign objects that block the sensor.
- Placed near other devices that may interfere with its operation.
- Excessive movement is present.
- Possible medical conditions that might interfere with readings.

**WARNING:** To ensure proper function of the Masimo Notification Escalation Policy (NEP), check the following:

- Do not disable the notification features on the Smart device (i.e. sounds, vibrations, etc.).
- Keep your smart device battery charged.
- Check your Smart device occasionally.

**CAUTION:** Do not place your smart phone where someone can change the App settings without you knowing.

**CAUTION:** Avoid using Masimo SafetyNet Alert under bright light sources and direct sunlight to maintain the performance of the device.

**CAUTION:** Check that the Masimo SafetyNet Alert alarm can be heard from other rooms in your home, especially when noisy appliances such as vacuum cleaners, dishwashers, clothes dryers, televisions, or radios are operating.

**CAUTION:** Keep the Bedside Station plugged in while in use. Loss of power may limit the notifications available in case of an event.

**CAUTION:** Only use the AC power supply that came with the Bedside Station to prevent damage to the device.

**CAUTION:** Do not connect to an electrical outlet controlled by a wall switch or dimmer.

**CAUTION:** When using Masimo SafetyNet Alert, locate the devices away from sources that may interfere with the Bluetooth connection. The presence of other devices that may create radio frequency interference (RFI) may result in loss of Quality of Service of the Bluetooth connection (see Specifications for details). Devices that may cause RFI include but are not limited to the following: cell phones, laptops and tablets, pagers, Bluetooth devices, devices with remote controls, electrocautery equipment, diathermy equipment, and baby monitors.

**CAUTION:** To ensure security and prevent tampering of your smart phone, follow the directions below:

- Smart device should be located with responsible users.
- Smart device should not be left unattended.
- Security features on smart device should be activated.
- Unauthorized changes should not be made to the Masimo SafetyNet Alert system.

**CAUTION:** Check your setup by viewing the display on the App and the Bedside Station. The App and Bedside Station will provide an indication if there is a problem with the internet connection.

**CAUTION:** Check your system setup by viewing the display on the smartphone Masimo SafetyNet Alert Application. The Masimo SafetyNet Alert App will provide an indication if there is a problem with the internet connection.

**CAUTION:** Avoid placing Masimo SafetyNet Alert near other wireless devices to prevent loss of performance.

## Masimo Sensor

**WARNING:** Properly apply the Masimo sensor according to sensor's Directions for Use. Applying the sensor incorrectly could result in incorrect or no readings.

## Cleaning and Service Warnings

**WARNING:** Do not attempt to remanufacture, recondition or recycle the Masimo sensor, Masimo chip or Bedside Station. to prevent harm or damage to the system.

**WARNING:** Always turn off and unplug the Bedside Station before cleaning to prevent harm or damage to the device.

**CAUTION:** Do not clean Masimo sensor, Masimo chip or Bedside Station with undiluted bleach, petroleum-based products, acetone, or other harsh solvents. Clean only with the solutions specified in this manual to prevent damage to the device.

**CAUTION:** Do not submerge Masimo sensor, Masimo chip or Bedside Station in liquid or attempt to sterilize by any method to prevent damage to the device.

## Compliance Warnings

**WARNING:** Changes or modifications not approved by Masimo can void the user's authority to operate the equipment.

**WARNING:** The frequency bands of this device (2.4 GHz and 5.15 to 5.25 GHz) are only for indoor use in accordance with international telecommunication requirements.

**WARNING:** Only use Masimo authorized devices with Masimo SafetyNet Alert. Using unauthorized devices with Masimo SafetyNet Alert may result in damage to the device and/or patient injury.

**WARNING:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Masimo SafetyNet Alert, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

**CAUTION:** Disposal of product: Comply with local laws when disposing the device and/or its accessories.

**CAUTION:** Do not place the Masimo SafetyNet Alert near electrical equipment that may affect the device, preventing it from working properly.

**CAUTION:** To minimize radio interference, other electrical equipment that emits radio frequency transmissions should not be in close proximity to Masimo SafetyNet Alert.

**CAUTION:** Keep the Masimo SafetyNet Alert away from electrical equipment that emits radio frequencies to minimize radio interference. Radio interference may result in no or inaccurate readings.

**Note:** Masimo SafetyNet Alert complies with the limits for a Class B digital device, per Part 15 of the FCC Rules. These limits were designed to provide reasonable protection against harmful interference in a residential installation. Masimo SafetyNet Alert generates, uses, and can radiate radio frequency energy and may cause interference with radio communications. To determine if

Masimo SafetyNet Alert interferes with radio or television reception, turn it off and see if the interference stops. To correct the interference, try the following:

- Adjust or move the receiver's antenna.
- Move the receiver farther away from Masimo SafetyNet Alert.
- Plug the receiver and Masimo SafetyNet Alert into outlets on different circuits.
- Consult the dealer or a radio/TV technician for help.

**Note:** This equipment has been tested and found to comply with the Class B limits for medical devices according to the IEC 60601-1-2: 2014, Medical Device Directive 93/42/EEC. These limits are designed to provide reasonable protection against harmful interference in all establishments, including domestic establishments.

**Note:** This device complies with part 15 of the FCC Rules and Industry Canada's license-exempt RSS standards. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**Note:** To satisfy RF exposure requirements, this device and its antenna must operate with a separation distance of at least 20 cm from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

**Note:** This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

**Note:** Users are advised that high-power radars are allocated as primary users (i.e. priority users) of the bands 5250-5350 MHz and 5650-5850 MHz and that these radars could cause interference and/or damage to LE-LAN devices.

**Note:** When using Masimo SafetyNet Alert consideration should be taken to local government frequency allocations and technical parameters to minimize the possibility of interference to/from other wireless devices.

# Risks and Benefits

---

## Your Body and Oxygen

When we breathe, oxygen moves in our blood around our body. When we are healthy or not fatigued, our lungs are able to bring the oxygen needed by our brain, organs, and tissue. As the state of our body is always changing, it is common to see continual fluctuations in your blood oxygen over time.

However, when we are sick, fatigued, or having trouble breathing, the body can require more oxygen than can be delivered. When this occurs, your blood oxygen level can drop. These changes cannot be seen, but if our oxygen gets too low, you may feel the effects such as being lightheaded, dizzy, or having difficulty breathing. If your oxygen level stays too low for too long, the lack of oxygen to your brain, organs, and tissue or hypoxia can have serious long-term effects to your health including death.

The Masimo SafetyNet Alert allows you to see your oxygen level so that you can understand how your body and oxygen are connected.

If you have a medical condition that effects your breathing (e.g., asthma, COPD), your normal blood oxygen level may be lower. For these conditions, it is important that you seek the advice of your physician.

For other conditions where your blood oxygen becomes low, it is good to remember that our blood oxygen is connected to our breathing and always changing.

If you have any concerns that your oxygen level is constantly decreasing, staying low, or you are not feeling well, it is very important to not self-diagnose and seek the advice of your physician.

When using any medical device, there are risks and benefits.

## Risks of Masimo SafetyNet Alert

- As with all medical electrical devices, there is always a risk of electrical, mechanical, and fire hazards. However, these risks have been mitigated through the design and testing of Masimo SafetyNet Alert.
- As with all devices with small parts, there is always a risk of a child's swallowing a component or choking. Keep small parts away from small children to prevent swallowing or choking.

- Items applied directly to the skin may cause skin irritation, pressure injury, or general discomfort. Periodically check the area where the Masimo sensor is applied to prevent potential irritations.
- As with all types of alarms, there is always a risk of missing or not hearing an alarm. To minimize this risk, ensure Masimo SafetyNet Alert is placed where it can be heard and the speaker is not blocked by other objects.
- As with all types of alarms, there is always a risk of false alarms. To minimize false alarms, ensure you follow all device and Masimo sensor directions, and follow troubleshooting instructions. Never assume an alarm is false; promptly address every alarm.

## Benefits of Masimo SafetyNet Alert

- Masimo SafetyNet Alert has the ability to monitor wirelessly without being tethered to a monitor. This allows users the freedom of movement during activities, including sleep, not conveniently possible with a cabled connection.
- Masimo SafetyNet Alert is equipped with audible and visual physiological alarms that can alert you to changes in the user's condition while monitoring within a home environment.
- Masimo SafetyNet Alert uses the same proven pulse oximetry used in hospitals shown to have fewer false alarms<sup>1</sup> compared to other brand technology.
- Masimo SafetyNet Alert allows for the storage and review of your blood oxygen level as you go about your normal activities, including exercise or sleep.

<sup>1</sup> *Barker S.J. Anesth Analg. 2002 Oct;95(4):967-72.*

# Description

---

## Masimo SafetyNet Alert System

The Masimo SafetyNet Alert system consists of the following:

Item	Description
1	Bedside Station
2	Smart phone with Masimo SafetyNet Alert App Installed *
3	Masimo Sensor
4	Masimo Chip

\* Smart phone not included.

# Features

The following describes the Masimo SafetyNet Alert system parts.

## Bedside Station Overview

The Bedside Station is a device that transfers measured data from the wireless Masimo Sensor to the Masimo Cloud. The Bedside Station uses wireless connectivity and provides audible alerts.

### 1. Pairing Symbol

Location on the Bedside Station for pairing the Masimo Chip.

### 2. Bedside Station Button

Used for pairing and silencing alarms.

### 3. Bluetooth Status LED

Shows the Bedside Station Bluetooth status. See *Bedside Station Lights* on page 17.

### 4. Bedside Station Status LED

Shows the Bedside Station power and monitoring status. See *Bedside Station Lights* on page 17.

### 5. Wi-Fi Status LED

Shows the Bedside Station Wi-Fi status. See *Bedside Station Lights* on page 17.

### 6. USB Power Connector

Power is provided to the Bedside Station from the AC adapter and a USB cable.

### 7. Masimo Chip Storage Tray

Location on the Bedside Station for storage of the chip when not in use.

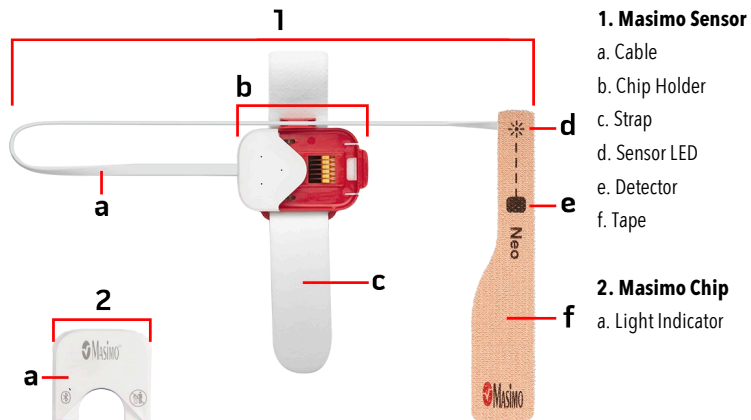
# Bedside Station Lights

The LED lights on the Bedside Station show the status of Bluetooth pairing and Wi-Fi connections.

Indicator/LED Color	What does it mean?
<b>Bedside Station Status LED</b>	
Solid White	Bedside Station is on and needs to be set up.
Solid Green	Bedside Station is paired with the sensor, connected to the Wi-Fi and communicating with the Masimo Cloud.
Flashing Orange	Active Level 1 Alert. See <i>Alarms and Notifications</i> on page 30.
Solid Orange	Level 1 Alert is acknowledged.
Flashing Red	Active Level 2 or 3 Alert. See <i>Alarms and Notifications</i> on page 30.
Solid Red	Level 2 or 3 Alert is acknowledged.
Flashing Orange	Sensor battery ports are blocked.
Flashing Red	A Bedside Station fault has been detected.
<b>Bluetooth Status LED</b>	
Solid White	Bedside Station is on and needs to be set up.
Off	Bedside Station is paired with the sensor.
Flashing White	Bedside Station is pairing with the sensor.
Solid Orange	Sensor is disconnected from the Bedside Station.
<b>Wi-Fi Status LED</b>	
Solid White	Bedside Station is on and needs to be set up.
Off	Bedside Station is connected to the Wi-Fi network.
Flashing White	Bedside Station is searching for or connecting to the Wi-Fi network.
Solid Orange	Bedside Station is connected to the Wi-Fi and the Sensor is no longer connected.

# Masimo Sensor and Chip Overview

The Masimo Sensor and Masimo Chip are for use with the Masimo SafetyNet Alert system.

## Sensor Lights

When inserted in the sensor, the light indicator on the Masimo Chip shows the status of the sensor.

Light Indicator Color	What does it mean?
Flashing Blue	Sensor is paired with the Bedside Station.
Flashing Purple	Sensor battery ports are blocked.
Flashing Green	Sensor is not paired with the Bedside Station.
Flashing Orange	Sensor battery is low.
Flashing Red	Sensor battery is very low.

# Basic Setup and Use

---

## Getting Started

The Masimo SafetyNet Alert system can be ready to use in the following steps:

1. Setup the Smart Phone - Prepare the smart phone for use with the Masimo SafetyNet Alert App.
2. Install the App – Download the Masimo SafetyNet Alert App on your smart phone and complete the registration and login process.
3. Setup the Masimo SafetyNet Alert System – Use the Masimo SafetyNet Alert App and complete the following:
  - Connect the Bedside Station using local Wi-Fi with the Masimo Cloud.
  - Pair the Masimo Sensor with the Bedside Station.
  - Select emergency contacts.
4. View Live Data – The Masimo SafetyNet Alert App and phone are ready for monitoring.

## Step 1: Smart Phone Setup

### Prepare the Smart Phone for Use

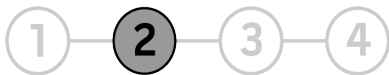
A compatible smart phone is required to install and operate the Masimo SafetyNet Alert App.

To use your smart phone with the Masimo SafetyNet Alert App check the following:

- **Compatibility**  
**Note:** For a list of smart phones and operating systems that work with the Masimo SafetyNet Alert App, check [www.mymasimo.co.uk/safetynet-alert](http://www.mymasimo.co.uk/safetynet-alert) before upgrading the smart phone or its operating system.
- **Smart Phone Battery is charged**
- **Bluetooth is ON**
- **Wi-Fi is ON and the smart phone is connected to the internet**

- **Time is set to the Current Local Time**

## Step 2: Install the Masimo SafetyNet Alert App

### Download and install the Masimo SafetyNet Alert App on the smart phone:

Download and install the Masimo SafetyNet Alert App. Follow the on-screen instructions to install. For more on how to install an app, see the smart phone's manual.

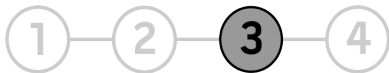
**Note:** If the Masimo SafetyNet Alert App requests for the smart phone to share its location, select *OK* or *Allow*. The smart phone location is required for Bluetooth connection.

1. After installation, open the Masimo SafetyNet Alert App and login to the user account
  - If you do not have an existing account, select **Sign Up**. Follow the on-screen instructions to create a new account and login.
2. The app walks you through the steps to set up Masimo SafetyNet Alert starting with the Bedside Station.

**Note:** Make sure your smartphone Bluetooth connection is turned on to connect to Masimo Sensor. Refer to the smart phone's instructions to change its settings.

## Step 3: Masimo SafetyNet Alert System Setup

After the Masimo SafetyNet Alert App is installed and setup, follow the app instructions to set up your Masimo SafetyNet Alert System:

- Pair the App to the Bedside Station.
- Connect the Bedside Station to the Wi-Fi.

- Setup the Masimo Sensor and pair to the Bedside Station.
- Add your Emergency Contacts.

## Bedside Station Setup

Follow the instructions below to set up the Masimo SafetyNet Alert system. The Bedside Station is set up in two (2) steps.

**Step 1:** Pair the Masimo SafetyNet Alert App to the Bedside Station using Bluetooth.

**Step 2:** Connect the Bedside Station to the wireless network (Wi-Fi).

**WARNING:** Place the Bedside Station in a safe location so as not to fall on anyone and where the alarm sounds can be heard.

1. Following the on-screen instructions, connect the Bedside Station to a power outlet using the AC power cord and adapter and select **Continue**.

**Note:** The Bedside Station Status LED is white when powered on.

2. Hold your smart phone near the Bedside Station during setup. When complete, the App screen will display: *Success*.  
**Note:** If the Bluetooth connection fails, hold the smart phone closer to the Bedside Station and try to reconnect. If the Bedside Station cannot connect, see **Troubleshooting** on page 41.
3. Next, connect to your Wi-Fi by selecting **Set Up Wi-Fi** and follow the app instructions.
4. Select the wireless *Network* and enter the *Password*. Select **Continue**.

**Note:** To view available Wi-Fi networks within range of the Bedside Station, click the *Network* field and select from the displayed list

5. Once the Wi-Fi connection is established, the *Bedside Station Wi-Fi Setup Successful* screen will appear. If the Bedside Station cannot connect, follow the on-screen instructions to verify the network and try again or view **Troubleshooting** on page 41.
6. Next, you will need to attach the Masimo Sensor and connect it with the Bedside Station. See **Masimo Sensor Setup** on page 22 for additional instructions.

## Masimo Sensor Setup

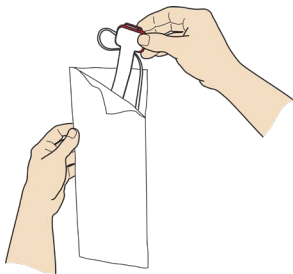
After the Bedside Station is set up, follow the on-screen instructions to:

- Set up the Masimo Sensor.
- Attach the sensor to your hand.
- Pair the sensor to the Bedside Station.
- Insert the Masimo Chip into the sensor.

## Attaching the Masimo Sensor


Follow the instructions to attach the Masimo Sensor to your hand.

1. Gather all components required for monitoring. See **Masimo Sensor and Chip Overview** on page 18.
2. Remove the sensor from the pouch.

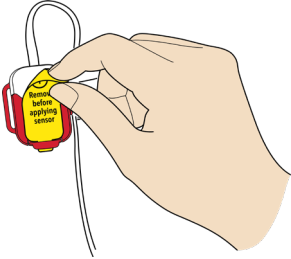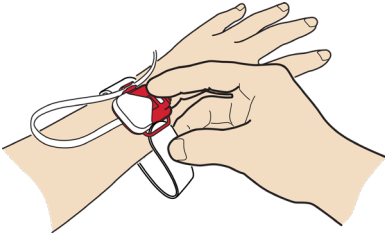
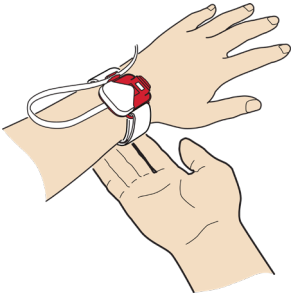
3. Peel off the yellow label to activate the battery.

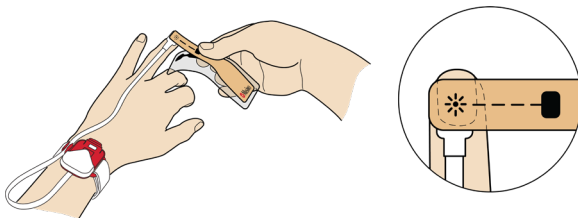
4. Place the sensor on the wrist as shown and carefully thread the tip of the strap through the open red c loop.


5. Wrap the loose attachment strap around the wrist and press to secure.


6. Remove part of the plastic film and place the star  on top of the fingernail of your ring finger.

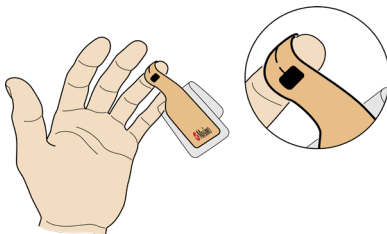
7. Wrap the tape around your finger so the square marking on the sensor tape is on the finger pad directly across from the star.

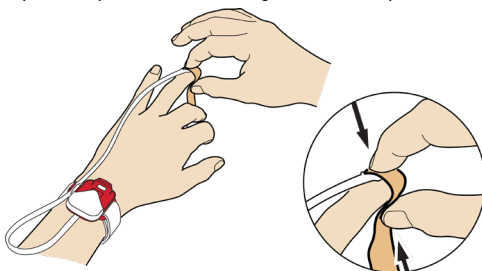
8. Squeeze top and bottom of finger to secure tape.

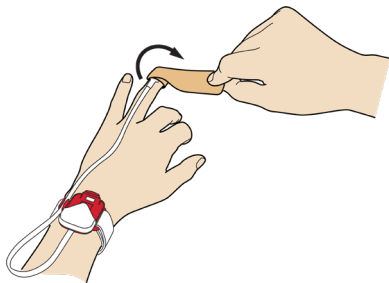9. Remove plastic film and continue to wrap remaining tape around your finger.

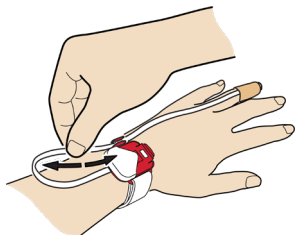
10. Adjust the sensor cable to a comfortable length. Go to ***Pairing the Masimo Sensor*** on page 26 for additional instructions.


## Pairing the Masimo Sensor

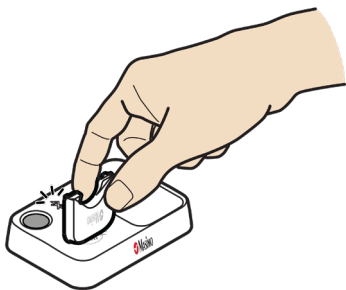
1. To pair the Masimo Sensor to the Bedside Station, hold the chip near the Bedside Station pairing symbol until an audible beep is heard and the Bedside Station Bluetooth LED flashes. See ***Bedside Station Overview*** on page 16.

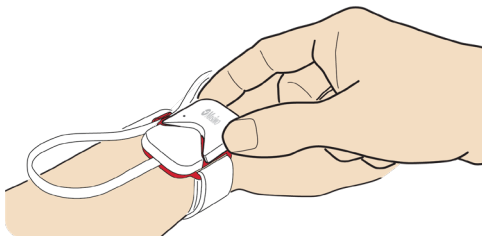2. Insert the Masimo Chip into open compartment on the sensor.

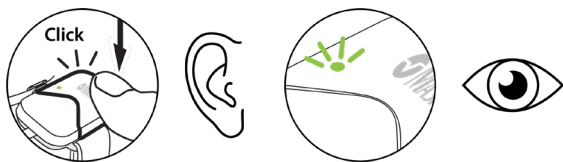3. Press down on the Masimo Chip to secure. Listen for a click and look for a green light on the chip to confirm successful chip connection to sensor.


4. Confirm successful pairing once the chip is inserted into the sensor by observing the light indicators.

#### **Bedside Station:**

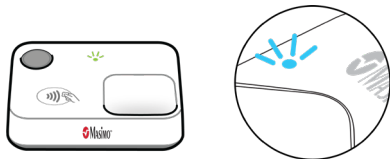- Bluetooth Status LED - turns off
- Status LED - turns green

#### **Masimo Chip:**

- Light Indicator - turns blue

- If the sensor and Bedside Station cannot pair, see **Troubleshooting** on page 41.

# Add Emergency Contacts

Follow the on-screen prompts to select your Emergency Contacts from your smart phone. This allows the Masimo SafetyNet Alert App to contact them in case of an emergency. An emergency contact is someone who will be alerted if you experience a low oxygen event

1. Choose *Add From Contacts* or *Add Manually* to get started.
2. If prompted to *Allow Masimo SafetyNet Alert to access your contacts*, select **Allow**.  
**Note:** If you **Deny** access, Masimo SafetyNet Alert cannot add contacts for use in case of an emergency.
3. Select your emergency contacts from your contact list or add manually by entering the contact name and mobile phone number.  
**CAUTION:** Do not use contacts with a landline phone number since they will not be able to receive text notifications in case of an emergency.
4. Select the types of alerts for the emergency contacts to receive and add the emergency contact.

Once contacts are added to receive alerts, the *Request Sent* screen appears. Select **Finish** to complete.

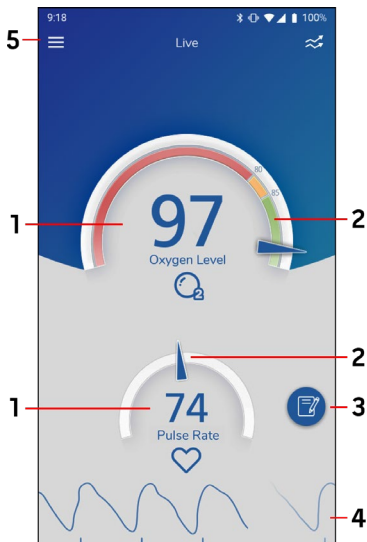
When finished, the *Live Data* screen appears. See **Step 4: View Live Data** on page 28.

- The emergency contact should receive a text message with a code. To accept, they must use the code and respond to the message as instructed.
- A text message is also sent to you when the emergency contact has accepted.

## Step 4: View Live Data

After setting up the Masimo SafetyNet Alert system, the app displays data from the Masimo Sensor. The *Live* screen is also the Masimo SafetyNet Alert App Main Screen, with access to other App functions.

## 1. Data Display

Displays the data from the sensor. Touch the display to show additional information. See **Data Information** on page 29.

## 2. Data Display Gauge

Displays the data from the sensor in a gauge style.

## 3. Timeline Notes Button

Touch to view and add Timeline notes. See **Timeline Notes** on page 36.

## 4. Pulse Rate Waveform Display

Displays the waveform of the pulse rate.

## 5. App Menu

Displays the *App Menu* screen.

\* If a sensor is not paired to the Bedside Station, the readings display as dashes and a pointer does not display on the gauge.

## Data Information

Selecting the measurement from the *Live* screen displays a pop-up window.

### Pulse Rate

Displays your Pulse Rate information.

### Oxygen Level

When selected, links to information about Oxygen Level are available.

- Select **Notifications** to view the Oxygen Level notifications.
- Select **Levels** to view Oxygen Levels.

# Alarms and Notifications

Masimo SafetyNet Alert provides visual and audible alarms when your oxygen level becomes low. Masimo SafetyNet Alert will also contact your emergency contacts and notify if your oxygen level drops too low and you are unaware or unable to respond.

## Alarm Levels and Messages

To help ensure your safety, a notification escalation policy can be activated to establish three (3) levels of notifications. The level of notification is based on your oxygen level and length of time at a low oxygen level.

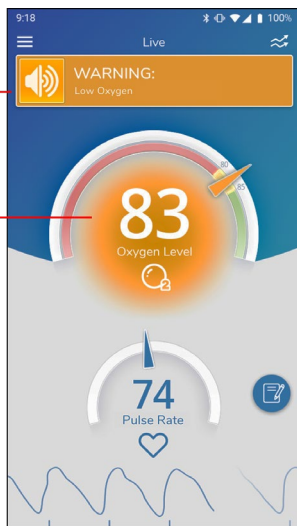
### Level 1

Alarm Trigger	Visual Alarm	Audible Alarm	Emergency Contact
Oxygen levels are less than 85% for 30 seconds.	Warning (Yellow)	Yes	Not Contacted

When Level 1 is triggered, an alarm sounds on your Bedside Station and the App. The App also displays:

1. Your Oxygen Level in the yellow zone.
2. A WARNING banner at the top of screen.

Touch the Oxygen Level **(1)** or WARNING banner **(2)** to learn what the Level 1 notification means.

## Alarm Information

Information about the low oxygen level event is shown along with steps on how to help your oxygen level recovery.

You have the option to:

**1.** Call a designated emergency contact **(1)** to assist you.

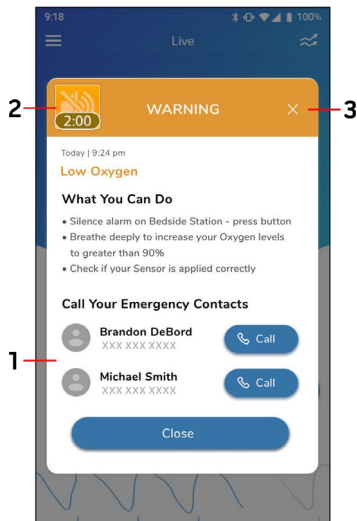
**2.** Silence the alarm

**On the App:** Tap the alarm icon on the top left corner of the pop-up **(2)**

**Bedside Station:** Press the button once.

See Silence Alarms for additional information.

To exit the screen, tap the "X" **(3)** or select the **Close** button.


## Levels 2 and 3

Notification Level	Trigger	Visual Alarm	Audible Alarm	Emergency Contact
Level 2	Oxygen levels are less than 80% for 30 seconds.	Emergency (Red)	Yes	Message sent to Emergency Contact
Level 3	Oxygen levels are less than 80% for 180 seconds or less than 60% for 60 seconds.	Emergency (Red)	Yes	Message sent to Emergency Contact

When Level 2 or 3 alarms are triggered, an alarm will sound on your Bedside Station and the App. The App will also display:

1. Your Oxygen Level in the red zone.
2. An EMERGENCY banner at the top of screen.
  - Level 2 banner message: *Very Low Oxygen*
  - Level 3 banner message: *Dangerously Low Oxygen*

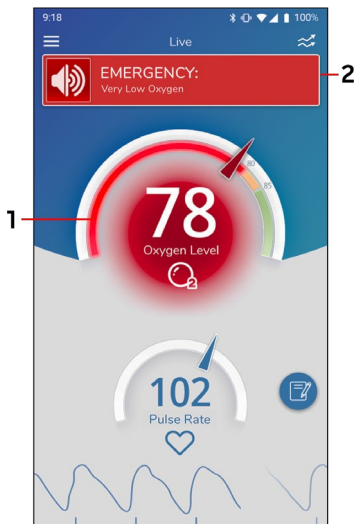Touch the Oxygen Level **(1)** or EMERGENCY banner **(2)** to learn what the Level 2 or 3 alarm means.

## Emergency Condition Alarm Information

Information about the low oxygen level event is given along with steps on how to help your oxygen level recovery.

Designated emergency contacts are notified to assist you.

You have the option to:

1. Silence the alarm

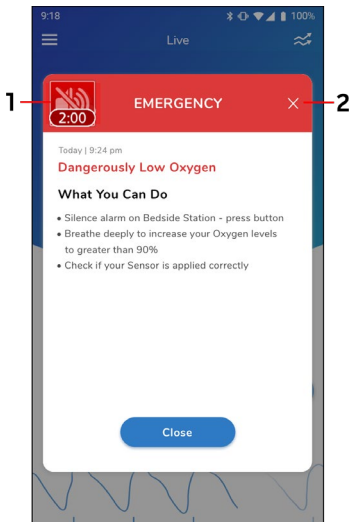
### Silence the Alarm

**On the App:** Tap the alarm icon on the top left corner of the pop-up **(1)**

**Bedside Station:** Press the button once.

See Silence Alarms for additional information.

To exit the screen, select the **"X" (2)** or select the **Close** button.


## Silence Alarms

An audio alarm can only be silenced for 2 minutes. After the 2 minutes, the alarm will sound again. You can continue to temporarily silence the alarm for 2 minutes at a time as long as the low oxygen level condition continues. If your oxygen level returns to a non-alarm level, the alarm will stop.


### Silence Alarms From the App

Touch the Speaker icon  shown on the banner to silence the App alarm.
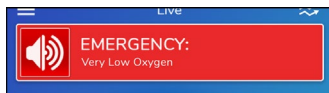
### Silence Alarms From the Bedside Station

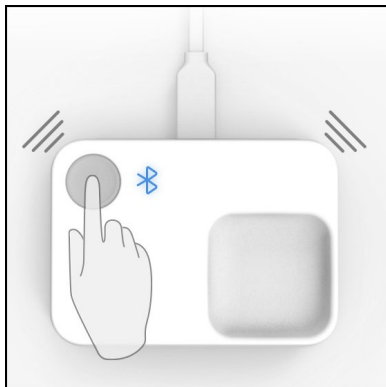Press and release the Bedside Station button to silence the Bedside Station alarm.

# Advanced Use

## History

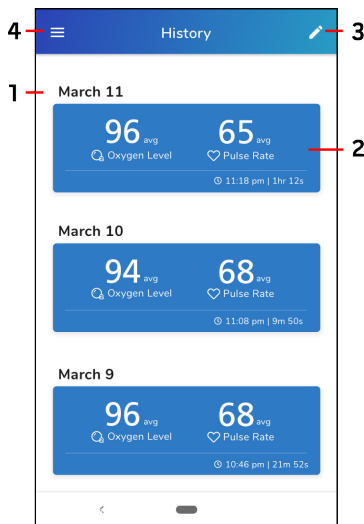
The *History* screen allows viewing of previous user sessions.

When no sessions are stored, the *History* screen is blank.

The features of the *History* screen are as follows:

1. Date of the session below.
2. History session information. Select to view this session.
3. Select the *Edit* icon to share, edit or delete one or more sessions.

To return to the *App Menu*, select the App Menu icon (4).

## Edit Session

When viewing the selected session, the Edit icon  at the top of the screen opens a menu with the following actions that be performed for this session:

- Share the session as an image.
- Delete this session.

Select **Cancel** to close the menu and return to the session screen.

## Share the Session as an Image

When **Share as Image** is selected, the Masimo SafetyNet Alert App uses the smart phone default email application to send the image. Enter the email addresses for contacts to receive an email with the image of the selected session.

## Delete the Session

When **Delete Session** is selected, a pop-up displays to confirm to **Delete** the session, or to **Cancel** deleting the session. When **Cancel** is selected, the screen returns to the session display.

## Timeline Notes

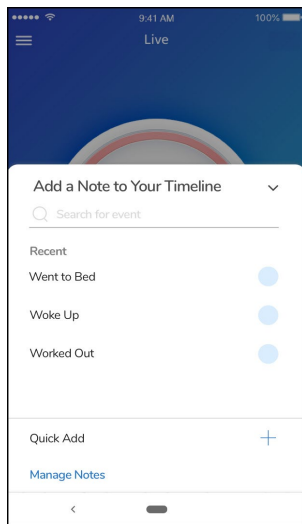
Notes can be added to the Timeline through the Notes menu.

Access the Notes menu while viewing the *Live* data screen by selecting the Notes icon .

The Notes menu then appears as shown.

## Exit

To collapse the notes menu, touch the down arrow in the upper right corner.



## Recent Notes

Notes that have been added previously are listed under *Recent*. You can select a note directly from the recent list.

After selecting the note from the *Recent* list, select **Save** to add the event.

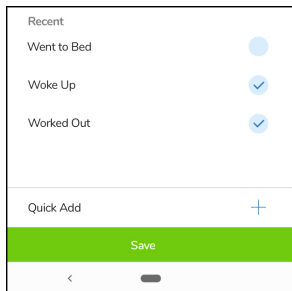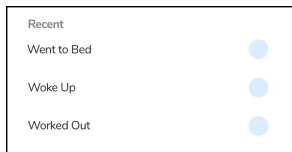**Note:** Multiple notes can be selected and added at the same time.


## Search for a Note

You can search for a note to add by typing the name of the note into **Search for event**.

After typing in the name of the note, select search. A list of related notes appear. Select the note from the list.

After adding the note, it appears under the *Recent* list.

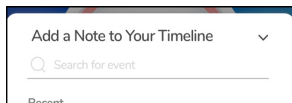Select the note, then **Save** to add the note to the Timeline.

## Add a New Note Using Quick Add

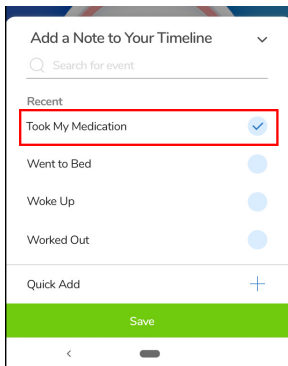
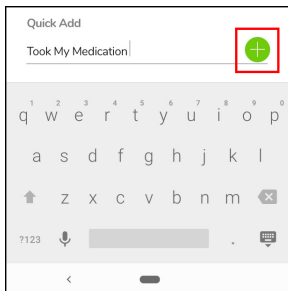
A new note that does not currently exist in the app can be manually added under *Quick Add*.

Select the "+" to the right, then type the name into the *Quick Add* field.

After typing in the name, select the *Quick Add* icon to add the note to the app.

After adding the note, it appears under the *Recent* list.

Select the note, then **Save** to add the *Quick Add* note to the Timeline.


## Manage Notes

Selecting *Manage Notes* from the Notes menu displays the *Manage Notes* screen. The *Manage Notes* screen displays the following features:


1. Displays list of available notes within the app.
2. Search for notes within the app.
3. Add a new note.
4. Delete a note.

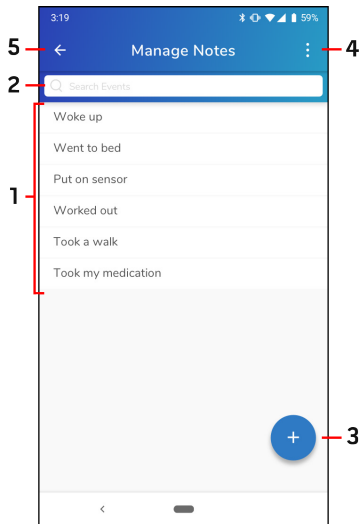To return to the previous screen, select the back arrow (5).

## Edit or Delete a Note

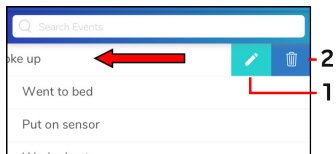
To edit or delete a note, swipe left on a note from the *Manage Notes* list and select the *Edit* (1) or the *Delete* (2) option.

## Edit a Note


When the Edit option is selected, the list of available notes display.

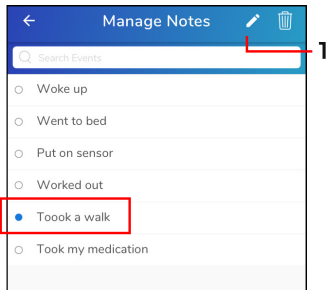Select the note from the list to be edited, then select the *Edit* icon (1) at the top of the screen.

The *Edit Note* window displays, allowing the name to be edited.

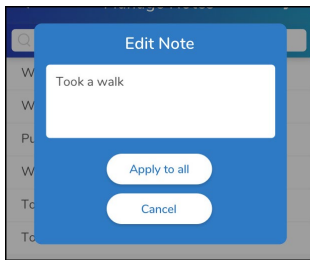After editing the note, select **Apply to all** to save the edits or **Cancel** to discard the changes.


## Delete a Note

When the Delete option is selected, the list of available manual notes display.

Select a single note (or multiple notes) from the list to be deleted, then select the *Delete* icon (1) at the top of the screen.

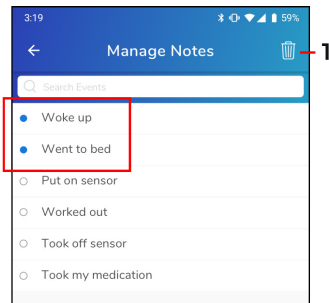**CAUTION:** You will not be prompted to confirm the deletion, and it cannot be undone.

# Troubleshooting

## Masimo SafetyNet Alert Messages

The following section lists possible messages, the potential cause, and next steps.

Displayed Messages	Potential Causes	Next Steps
<i>"Difficulty in obtaining a reading"</i>	Interference while monitoring.	Please ensure that: <ol style="list-style-type: none"><li>1. The sensor is not on a finger with jewelry, such as a ring.</li><li>2. You move away from ambient lighting and computer displays/TVs.</li><li>3. You are not wearing artificial nails or excessive nail polish on the monitored finger.</li></ol> If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.
<i>"Replace your sensor"</i>	Sensor is not working.	Replace sensor.
<i>"Wireless sensor disconnected during an alert"</i>	Sensor became disconnected during audible alerts for including: <ul style="list-style-type: none"><li>• Physiology alert or technical alert</li><li>• Low SpO<sub>2</sub></li><li>• Sensor off</li><li>• Obstructed battery</li><li>• Low battery</li><li>• Depleted battery</li></ul>	<ul style="list-style-type: none"><li>• Press the alarm silence button on the Bedside Station. Follow the instructions and attempt to pair the sensor again. See <b>Pairing the Masimo Sensor</b> on page 26.</li><li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li></ul>
<i>"Allow Masimo SafetyNet Alert to access your contacts"</i>	Access to contacts on smart phone has not been granted.	Allow smart phone permissions to share contacts with the Masimo SafetyNet AlertMasimo SafetyNet Alert App.

Displayed Messages	Potential Causes	Next Steps
<i>"Your Emergency Contact list is empty. Touch the plus icon to start adding Emergency Contacts"</i>	When emergency contact list does not include a contact.	Follow instructions on the app to add emergency contacts.
<i>"Cannot modify a device that is in an active monitoring state. Please end your monitoring session and try again."</i>	When trying to change a setting that may disrupt alarms/notifications while in a monitoring state.	End the monitoring session before attempting to modify the device.
<i>"Confirm you Email"</i>	When an unverified email address was used to log into the app.	Confirm the email as requested.
<i>"A confirmation email was sent to emailaddress@example.com. Follow the instructions for confirming your email address then return to the App."</i>	When a new email address is submitted during setup.	Follow the instructions on the confirmation email.
<i>"An Emergency Contact is a designated friend, family member or caregiver that will be alerted in the event you need immediate assistance regarding your Oxygen levels. SafetyNet gives you the option to Add an Emergency Contact now or later."</i>	When adding an emergency contact to describe an emergency contact.	<ul style="list-style-type: none"> <li>• For information only, no action is required.</li> <li>• For more information on emergency contacts go to <b><a href="https://my.masimo.co.uk/safetynet-alert">my.masimo.co.uk/safetynet-alert</a></b>.</li> </ul>
<i>"Add Manually" "Create a new emergency contact. Please fill out the following information below"</i>	When adding an emergency contact manually.	For information only, no action is required.

Displayed Messages	Potential Causes	Next Steps
<p><i>"Request Sent"</i>  <i>"A request has been sent to your Emergency Contact. Follow up with your Emergency Contact to make sure they accept."</i></p>	<p>Displayed after successfully sending an emergency contact invitation</p>	<p>For information only, no action is required.</p>
<p><i>"Email Sent!"</i>  <i>"Please check your email to confirm"</i></p>	<p>Resend button was pressed to request a new email verification link.</p>	<p>For information only, no action is required.</p>
<p><i>"Device in use"</i>  <i>"Cannot modify a device that is actively monitoring. Please end your monitoring session and try again."</i></p>	<p>When an attempt was made to modify or delete the Bedside Station during active monitoring.</p>	<p>End the monitoring session before attempting to modify the device.</p>
<p><i>"Remove this emergency contact?"</i>  <i>"Remove selected emergency contacts?"</i></p>	<p>Displayed when removing one or more emergency contacts.</p>	<p>Select Remove to confirm or Cancel to cancel the operation.</p>
<p><i>"Remove this device?"</i></p>	<p>Displayed when attempting to remove Bedside Station</p>	<p>Select Remove to confirm or Cancel to cancel the operation.</p>
<p><i>"Save Changes?"</i>  <i>"Your unsaved changes will be lost. Save changes before closing?"</i></p>	<p>Displayed when attempting to update profile information.</p>	<p>Select Yes to confirm or No to not save changes.</p>
<p><i>"Your Bedside Station is paired"</i></p>	<p>The Bedside Station has paired successfully.</p>	<p>For information only, no action is required.</p>
<p><i>"No Bedside Station added"</i></p>	<p>Bedside Station is not connected to the app.</p>	<p>Follow instruction on the app or this manual to complete set up. See <b>Bedside Station Setup</b> on page 21.</p>

Displayed Messages	Potential Causes	Next Steps
<i>"Bedside Station not found"</i>	Timeout has occurred during setup when searching for Bedside Station.	Follow the troubleshooting section of this manual to resolve the issue.
<i>"Bedside Station Disconnected from Server"</i>	Bedside Station is disconnected from the cloud server.	Follow the troubleshooting section of this manual to resolve the issue.
<i>"Bedside Station Error"</i>	Bedside Station internal hardware fault.	The Bedside Station requires replacement, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.
<i>"Unable to connect to Wi-Fi"</i>	Wi-Fi connection failure due to timeout or invalid password.	Enter the correct password.
<i>"No Internet Connection"</i>	<ul style="list-style-type: none"> <li>The smart phone is not connected to a Wi-Fi network.</li> <li>The smart phone is not connected to a cellular network.</li> </ul>	Ensure the smart phone is connected to a Wi-Fi or cellular network.
<i>"Enable phone Bluetooth"</i>	Smart phone Bluetooth is turned off.	Turn on the smart phone Bluetooth.
<i>"The username entered already exists, please try another"</i>	User name already exist with another user.	Please choose another user name.
<i>"Battery low Warning"</i>	Sensor battery is low.	Replace sensor.
<i>"Depleted Battery"</i>	Sensor battery is depleted.	Replace sensor.
<i>"Wireless Sensor Disconnected"</i>	Sensor is not monitoring.	Ensure sensor placement steps are completed and you are obtaining readings.

Displayed Messages	Potential Causes	Next Steps
<i>"Your oxygen level measurement has a low value of 76%"</i>	Level 1 alarm is triggered.	Follow instruction on the app to resolve the condition.
<i>"Obstructed Battery Port"</i>	Sensor is not monitoring.	Please keep the sensor battery clear of any direct contact.
<i>"Place Sensor on Properly"</i>	Sensor is not monitoring.	Place the sensor on properly. You may go to the options menu to view the sensor placement videos. See <b>Masimo Sensor Setup</b> on page 22.
<i>"Allow Masimo SafetyNet Alert to access this device's location"</i>	Location services permissions have not yet been granted when attempting to search for a Bluetooth device.	Allow the smart phone permission to share its location with the Masimo SafetyNet AlertMasimo SafetyNet Alert App.
<i>"Please Enable App Notifications"</i>	Notifications are not enabled on the smart phone for the Masimo SafetyNet Alert App.	Enable notifications on the smart phone for the Masimo SafetyNet Alert App.
<i>"Low Disc Space Warning!"</i>	Low diskspace during active monitoring and writing session trend.	Clear diskspace on the smart phone. A minimum of 100MB free disc space is required on the smart phone for the Masimo SafetyNet Alert App to operate properly.
<i>"You cannot access this item at the moment."</i>	Error when connecting to the server.	<ul style="list-style-type: none"> <li>• Try again.</li> <li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>"There was an error processing your request."</i>	Error when connecting to the server.	<ul style="list-style-type: none"> <li>• Try again.</li> <li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>"This request is not understood."</i>	Error when connecting to the server.	<ul style="list-style-type: none"> <li>• Try again.</li> <li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>

Displayed Messages	Potential Causes	Next Steps
<i>"Username or password invalid, please try again."</i>	Error when connecting to the server. Displayed when invalid credentials are used during the sign in process	<ul style="list-style-type: none"> <li>• Reenter the Username and Password.</li> <li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>"The username entered already exists, please try another."</i>	Error when connecting to the server. Displayed during registration when the username/email is already in use by another user.	Try a new Username and Password.
<i>"This account has already been verified."</i>	When attempting to resend an email Verification link to an already verified account.	Continue with Masimo SafetyNet Alert App setup, as email has already been verified.
<i>"There is no account registered under this username."</i>	Error when connecting to the server. Displayed when a user attempts to sign in with a username that does not exist.	Reenter the Username and Password. If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.
<i>"User has already registered a different device"</i>	Error adding a new device. When adding a device fails due to different existing device	Follow steps to remove current device in order to add new device.
<i>"This bedside device has already been registered to an account."</i>	Error adding a device. When attempting to register a device that has already been registered to another user	Remove the device from the original account. If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.
<i>"Device deleted successfully"</i>	When attempting to remove a device that no longer exists for this user on the cloud.	For information only, no action is required.
<i>"Invitation already sent to this Emergency Contact."</i>	When attempting to add an emergency contact that has a pending/active invitation with the current user.	For information only, no action is required.

Displayed Messages	Potential Causes	Next Steps
"Emergency Contact deleted successfully"	When attempting to remove an emergency contact that no longer exists for this user on the cloud.	For information only, no action is required.
"Maximum number of emergency contacts reached."	When attempting to add an emergency contact when the limit has reached. Limit = 10	For information only, no action is required.
"Something unexpected happened. Please try again later."	System error.	<ul style="list-style-type: none"> <li>• Try again.</li> <li>• If you still experience issues, please contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>

## Troubleshooting Masimo SafetyNet Alert

The following section lists possible symptoms, the potential cause, and next steps.

Symptom	Potential Causes	Next Steps
<i>Masimo SafetyNet Alert App does not turn on</i>	<ul style="list-style-type: none"> <li>• Depleted smart phone battery.</li> <li>• Masimo SafetyNet Alert App needs to be updated.</li> <li>• Incompatible smart phone.</li> </ul>	<ul style="list-style-type: none"> <li>• Connect smart phone to battery charger and charge battery. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check for Masimo SafetyNet Alert App updates.</li> <li>• Check smart phone compatibility. See <b>Specifications</b> on page 53.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>Masimo SafetyNet Alert App turns off</i>	<ul style="list-style-type: none"> <li>• Depleted smart phone battery.</li> <li>• Masimo SafetyNet Alert App needs to be updated.</li> <li>• Incompatible smart phone.</li> </ul>	<ul style="list-style-type: none"> <li>• Connect smart phone to battery charger and charge battery. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check smart phone compatibility. See <b>Specifications</b> on page 53.</li> <li>• Check for Masimo SafetyNet Alert App updates.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>

Symptom	Potential Causes	Next Steps
<p><i>Masimo SafetyNet Alert App does not communicate with Bedside Station</i></p>	<ul style="list-style-type: none"> <li>• Bedside Station is not powered on.</li> <li>• Smart phone is not in close proximity to Bedside Station.</li> <li>• Bluetooth on the smart phone is not turned on and/or not correctly configured.</li> <li>• Smart phone does not support Bluetooth Low Energy (BLE).</li> </ul>	<ul style="list-style-type: none"> <li>• Check that Bedside Station is plugged into the AC power supply.</li> <li>• Ensure the smart phone is in close proximity with Bedside Station.</li> <li>• Ensure Bluetooth on the smart phone is turned on.</li> <li>• Check smart phone compatibility. See <b><i>Specifications</i></b> on page 53.</li> <li>• Update the smart phone software. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Contact Masimo Customer Support. See <b><i>Customer Support</i></b> on page 69.</li> </ul>
<p><i>Bedside Station does not connect to Wi-Fi or Masimo Cloud</i></p>	<ul style="list-style-type: none"> <li>• Bedside Station is not plugged in.</li> <li>• Incorrect Wi-Fi network selected.</li> <li>• Incorrect Wi-Fi password is entered.</li> <li>• Wi-Fi network is not correctly configured.</li> <li>• Masimo Cloud may be down.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure Bedside Station is plugged in.</li> <li>• Ensure smart phone is connected to correct Wi-Fi network. See <b><i>Bedside Station Setup</i></b> on page 21.</li> <li>• Ensure correct Wi-Fi network is selected. See <b><i>Bedside Station Setup</i></b> on page 21.</li> <li>• Ensure correct Wi-Fi password is entered. See <b><i>Bedside Station Setup</i></b> on page 21.</li> <li>• Check that the wireless features are correctly configured. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check network settings and availability. You may need to call network provider for further assistance.</li> <li>• Contact Masimo Customer Support. See <b><i>Customer Support</i></b> on page 69.</li> </ul>

Symptom	Potential Causes	Next Steps
<p><i>Masimo Sensor does not pair with Bedside Station</i></p>	<ul style="list-style-type: none"> <li>• Bedside Station is not plugged in.</li> <li>• Chip is not inserted into sensor.</li> <li>• Sensor is not in close proximity with Bedside Station during pairing.</li> <li>• Pairing button is released too soon.</li> <li>• Incorrect user logged into the Masimo SafetyNet Alert App.</li> <li>• Bedside Station has been registered with another account.</li> <li>• Depleted sensor battery.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure Bedside Station is plugged in.</li> <li>• Ensure the chip is firmly inserted into the sensor. See <b><i>Attaching the Masimo Sensor</i></b> on page 22.</li> <li>• Ensure the sensor is in close proximity with Bedside Station during pairing.</li> <li>• Ensure to press and hold the pairing button for 5 seconds until an audible tone is heard. See <b><i>Bedside Station Setup</i></b> on page 21.</li> <li>• Ensure correct user is logged into the Masimo SafetyNet Alert App.</li> <li>• Contact Masimo Customer Support. See <b><i>Customer Support</i></b> on page 69.</li> </ul>
<p><i>Bedside Station turns off (not lit up)</i></p>	<ul style="list-style-type: none"> <li>• Bedside Station is not plugged in.</li> <li>• Internal components may not be working properly.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure Bedside Station is plugged in.</li> <li>• Contact Masimo Customer Support.</li> </ul>
<p><i>Bedside Station speaker does not work (no sound or muffled sound)</i></p>	<ul style="list-style-type: none"> <li>• Speaker may be blocked by environment (ex. blankets or other appliances).</li> <li>• Internal components may not be working properly.</li> </ul>	<ul style="list-style-type: none"> <li>• Turn Bedside Station on and off by unplugging the unit.</li> <li>• Check that the device speaker is not being muffled.</li> <li>• Check that the Bedside Station is on a flat surface with minimal objects surrounding it.</li> <li>• Contact Masimo Customer Support. See <b><i>Customer Support</i></b> on page 69.</li> </ul>

Symptom	Potential Causes	Next Steps
<i>Masimo Cloud unavailable</i>	<ul style="list-style-type: none"> <li>• Wi-Fi is not turned on and/or not correctly configured.</li> <li>• Wireless service is weak or unavailable in the current location.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure the smart phone is within range of the wireless network for connection to the system.</li> <li>• Check that the wireless feature for smart phone is on and correctly configured. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check Wi-Fi network settings and availability.</li> <li>• Check wireless availability for location.</li> <li>• Update the smart phone software. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>Incorrect user data is displayed</i>	<ul style="list-style-type: none"> <li>• Incorrect user currently logged into the Masimo SafetyNet Alert App.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure the correct user is logged into the Masimo SafetyNet Alert App.</li> <li>• Log out the incorrect user and log into the correct account.</li> <li>• Restart SafetyNet app and login to the system.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>NO user data is displayed</i>	<ul style="list-style-type: none"> <li>• Incorrect user logged into the Masimo SafetyNet Alert App.</li> <li>• Wi-Fi is not correctly configured.</li> <li>• Smart phone settings are incorrect.</li> <li>• Sensor is not connected to the Bedside Station.</li> <li>• No previous sessions have been recorded.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure the correct user is logged into the Masimo SafetyNet Alert App.</li> <li>• Restart Masimo SafetyNet Alert App and login to the system.</li> <li>• Check that the wireless feature is correctly configured. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check network settings and availability. May have to call network provider for further assistance.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>

Symptom	Potential Causes	Next Steps
<i>Delayed data updates</i>	<ul style="list-style-type: none"> <li>• Wi-Fi is not correctly configured.</li> </ul>	<ul style="list-style-type: none"> <li>• Check that the wireless feature is correctly configured. Refer to the smart phone's Operator's Manual or Directions For Use.</li> <li>• Check network settings and availability. You may need to call network provider for further assistance.</li> <li>• Restart Masimo SafetyNet Alert App and login to the system. See</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>User alarms do not appear during events</i>	<ul style="list-style-type: none"> <li>• Incorrect user currently logged into Masimo SafetyNet Alert App.</li> <li>• Wi-Fi network not available.</li> <li>• System settings have changed.</li> </ul>	<ul style="list-style-type: none"> <li>• Ensure Masimo SafetyNet Alert is connected to Wi-Fi. See <b>Bedside Station Setup</b> on page 21.</li> <li>• Ensure the correct user is logged into Masimo SafetyNet Alert App.</li> <li>• Restart Masimo SafetyNet Alert and login to the system.</li> <li>• Ensure alert notification settings are turned on.</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>
<i>Masimo SafetyNet Alert App does not detect that the sensor is applied to the hand</i>	<ul style="list-style-type: none"> <li>• Sensor not properly placed on user.</li> <li>• Sensor not properly paired to Bedside Station.</li> <li>• Damaged sensor.</li> <li>• Internal failure.</li> </ul>	<ul style="list-style-type: none"> <li>• Reapply the sensor.</li> <li>• Pair the sensor with Bedside Station. See <b>Pairing the Masimo Sensor</b> on page 26.</li> <li>• Replace sensor.</li> <li>• Turn Bedside Station Off and On by unplugging the device then plug it in to start up</li> <li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li> </ul>

## Troubleshooting Measurements

The following section lists possible measurement symptoms, potential causes, and next steps. For more information, see **Safety Information** on page 6.

Symptom	Potential Causes	Next Steps
<i>Difficulty obtaining a reading.</i>	<ul style="list-style-type: none"><li>• Incorrect placement of sensor on user.</li><li>• Misalignment of sensor components.</li><li>• Low perfusion (blood flow).</li><li>• Excessive user motion.</li><li>• Excessive ambient or strobing light.</li><li>• Low battery/SafetyNet Bedside Station not plugged into AC power supply.</li></ul>	<ul style="list-style-type: none"><li>• Check the placement and alignment of the sensor on the hand. Re-apply sensor or move to a different location.</li><li>• Allow time for the parameter measurement to stabilize.</li><li>• Check if blood flow to the sensor location is restricted. Warm the hand where the sensor is placed.</li><li>• Minimize or eliminate motion at the monitoring location.</li><li>• Shield the sensor from excessive or strobing light.</li><li>• Replace sensor.</li><li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li></ul>
<i>Measurement values displayed as dashes.</i>	<ul style="list-style-type: none"><li>• Measurement may still be in progress.</li><li>• Incorrect placement of sensor on user.</li><li>• No Connection to Masimo Cloud.</li><li>• Sensor is damaged, not functioning or has a dead battery.</li></ul>	<ul style="list-style-type: none"><li>• Allow time for the parameter measurement to stabilize.</li><li>• Check the placement of the sensor on the hand. Re-apply sensor or move to a different location.</li><li>• Check if blood flow to the sensor location is restricted.</li><li>• Replace sensor.</li><li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li></ul>
<i>Unexpected or unlikely measurement values</i>	<ul style="list-style-type: none"><li>• Incorrect placement of sensor on user.</li><li>• Low signal quality.</li></ul>	<ul style="list-style-type: none"><li>• Check the placement of the sensor on the hand. Re-apply sensor or move to a different location.</li><li>• Move the sensor to a location on the user's body with stronger blood flow. Warm the hand where the sensor is placed.</li><li>• Contact Masimo Customer Support. See <b>Customer Support</b> on page 69.</li></ul>

# Appendix

---

## Specifications

### Masimo SafetyNet Alert App

#### Measurement Range

Measurement	Display Range	Unit of Measure
Oxygen Level (SpO <sub>2</sub> )	0 to 100	%
Pulse Rate	0 to 240	bpm

#### Smart Phone Compatibility

Item	Specification*
Operating System	Android 6.0 (Marshmallow) (minimum)
	iOS 12.0 (minimum)

\* For complete specifications, refer to [www.mymasimo.co.uk/safetynet-alert](http://www.mymasimo.co.uk/safetynet-alert).

### Masimo Sensor

#### Weight Range and Measurement Site

Population	Weight	Measurement Site
Adult and Pediatric	> 40 kg (> 88.2 lbs)	Finger
Neonate	< 3 kg (< 6.6lbs)	Finger or Foot

## Accuracy (ARMS\*)

Oxygen Level (SpO <sub>2</sub> )		
Range	70% to 100%	
No Motion [1]	Adults, Pediatrics	2%
	Neonates	3%
Motion [2]	Adults, Pediatrics, Neonates	3%
Low perfusion [3]	Adults, Pediatrics	2%
	Neonates	3%
Pulse Rate (PR)		
Range	25 bpm to 240 bpm	
No motion	Adults, Pediatrics, Neonates	3 bpm
Motion	Adults, Pediatrics, Neonates	5 bpm
Low Perfusion [4]	Adults, Pediatrics, Neonates	3 bpm

\*  $A_{RMS}$  accuracy is a statistical calculation of the difference between device measurements and reference measurements. Approximately two-thirds of the device measurements fell within  $\pm A_{RMS}$  of the reference measurements in a controlled study.

## Oxygen Level (SpO<sub>2</sub>) Performance Specifications

The tables below provides  $A_{RMS}$  (Accuracy Root Mean Square) values measured using the Radius PPG sensors under no motion, with Masimo Technology in a clinical study.

Measurement $A_{RMS}$ Values for Radius PPG Sensors	
SpO <sub>2</sub> Accuracy Range (%)	ARMS (%)
90-100	1.14 (1,14)
80-90	1.29 (1,29)
70-80	1.14 (1,14)

Measurement $A_{RMS}$ Values for Radius PPG Sensors	
SpO <sub>2</sub> Accuracy Range (%)	ARMS (%)
70-100	1.33 (1,33)

The below Bland-Altman plot represents the correlation of the  $(SpO_2 + SaO_2)/2$  versus  $(SpO_2 - SaO_2)$  under no motion with an upper 95% and lower 95% limits of agreement.

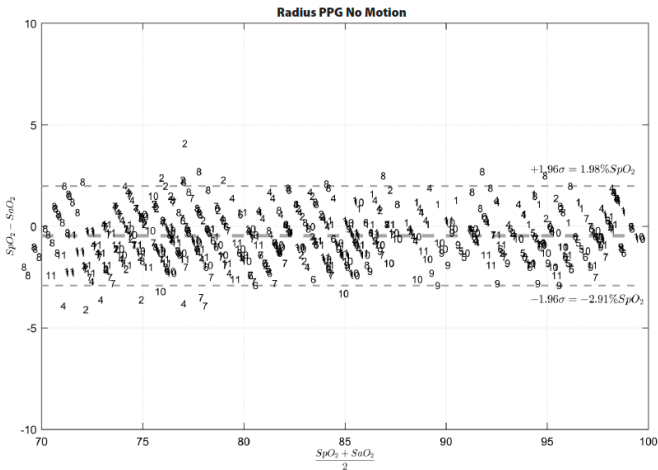
Figure 1: Radius PPG Sensors (ARMS 70-100%)

## Electrical

Battery - Masimo Sensor	
Run Time	96 hours in typical continuous usage

## Environmental

<b>Masimo Sensor Environmental Conditions</b>	
Operating Temperature	32°F to 104°F (0°C to 40°C)
Storage Temperature	32°F to 122°F (0°C to 50°C)
Operating Humidity	5% to 95%, non-condensing
Storage Humidity	5% to 95%, non-condensing

## Masimo Chip

### Environmental

<b>Masimo Chip Environmental Conditions</b>	
Operating Temperature	32°F to 104°F (0°C to 40°C)
Storage Temperature	-40°F to 158°F (-40°C to 70°C)
Operating Humidity	5% to 95%, non-condensing
Storage Humidity	5% to 95%, non-condensing
Atmospheric Pressure	540 to 1060 mBar @ ambient temperature and humidity

## Bedside Station

### Electrical

<b>AC Power Requirements - Bedside Station</b>	
AC Power Input	100 to 240 VAC, 50 to 60 Hz
DC Power Input	5V

## Physical Characteristics

<b>Bedside Station</b>	
Dimensions	3.45" x 2.48" x 0.78" (87.63 mm x 62.99 mm x 19.81 mm)
Weight	0.12 lbs. (56g)

## Environmental

<b>Bedside Station Environmental Conditions</b>	
Operating Temperature	41°F to 104°F (5°C to 40°C)
Storage Temperature	-13°F to 158°F (-25°C to 70°C)
Operating Humidity	10% to 95%, non-condensing
Storage Humidity	10% to 95%, non-condensing

## Compliance

<b>EMC Compliance</b>
IEC 60601-1-2:2014, Class B

<b>Safety Standards Compliance</b>
IEC 60601-1
IEC 62304
IEC 60601-1-11

<b>Equipment Classification per IEC 60601-1</b>	
Type of Protection	Class II (AC Power)

<b>Equipment Classification per IEC 60601-1</b>	
Degree of Protection against Electrical Shock	Type BF-Applied Part
Protection against harm from Water and Particulate Matter	IP22 (Protection from solid foreign objects $\geq 12.5$ mm diameter and against ingress from vertically falling water drops when enclosure tilted up to $15^\circ$ )
Mode of Operation	Continuous

## Wireless Specifications

<b>Communication (Bluetooth)</b>	
Type	Bluetooth
Frequency	2402-2480 MHz
Max Peak Output Power	8.45 dBm
Classification of Output Power Rating	Conducted
Output Power Type	Fixed at the Factory
Modulation Types	GFSK
Modulation Signals	Analog and Digital
Available Data Rates	1 Mbps

<b>Communication (Wi-Fi)</b>	
Type	WLAN Radio: IEEE 802.11 b/g/n
Frequency	802.11b/g/n(HT20): 2412-2462 MHz 802.11n(HT40): 2422-2452 MHz
Max Peak Output Power	WLAN 27.12 dBm
Classification of Output Power Rating	Conducted
Output Power Type	Fixed at the Factory

<b>Communication (Wi-Fi)</b>	
Modulation Types	802.11b: DSSS 802.11g/n(HT20/HT40): OFDM
Modulation Signals	Analog and Digital
Available Data Rates	802.11b - 1, 2, 5.5, 11 Mbps. 802.11g - 6, 9, 12, 18, 24, 36, 48, 54 Mbps 802.11n- MCS0 – MCS7

<b>Security and Authentication</b>	
Encryption	64/128-bit WEP, Dynamic WEP, WPA-TKIP, WPA2-AES
Authentication	Open System, Shared Key, Pre-Shared Key (PSK), 802.1X: LEAP, PEAP, TTLS, TLS, EAP-FAST

<b>Radio Compliance</b>	
USA	Contains FCC ID: 2AC7Z-ESP32SOLO1 FCC ID: VKF-CONNHUB
Canada	Contains IC: 21098-ESP32SOLO1 IC: 7362A-CONNHUB
Europe	EU Radio Equipment Directive (RED 2014/53/EU) EN 300 330 V2.1.1:2017 EN 301 489-3 V2.1.1:2019 1999/519/EC EN 62311:2020

# Guidance and Manufacturer's Declarations - Electromagnetic Compliance

## Electromagnetic Emissions

<b>Electromagnetic Emissions</b>		
The ME Equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the ME Equipment should assure that it is used in such an environment.		
<b>Emission Test</b>	<b>Compliance</b>	<b>Electromagnetic Environment - Guidance</b>
RF Emissions (Radiated) CISPR 11	Group 1 Class B	ME Equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions (Conducted) CISPR 11	Group 1 Class B	Suitable for use in all establishments, including domestic environments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Complies	

# Electromagnetic Immunity

<b>Electromagnetic Immunity</b>			
The ME Equipment is intended for use in the electromagnetic environment specified below. The customer or the user of the ME Equipment should assure that it is used in such an environment.			
<b>Immunity Test</b>	<b>IEC 60601 Test Level</b>	<b>Compliance Level</b>	<b>Electromagnetic Environment - Guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	+/- 8 kV contact +/- 15 kV air	+/- 8 kV contact +/- 15 air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/ burst IEC 61000-4-4	+/- 2 kV for power lines +/- 1 kV for input/ output lines	+/- 2 kV for power lines +/- 1 kV for input/ output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	+/-1 kV line(s) to line(s) +/- 2 kV line(s) to earth	+/-1 kV line(s) to line(s) +/- 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 Vrms	3 Vrms	Performed over 0.15-80 MHz
	6 Vrms	6 Vrms	Performed on the following ISM (industrial, scientific and medical) bands of frequency: The bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz

## Electromagnetic Immunity

Power frequency (50 / 60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of typical location in a typical hospital environment.
Voltage dips on power supply input lines IEC 61000-4-11	0% $U_T$ , 0.5 cycle, at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°; 0% $U_T$ 1 cycle, and 70% $U_T$ 25/30 cycles at 0°	0% $U_T$ , 0.5 cycle, at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°; 0% $U_T$ 1 cycle, and 70% $U_T$ 25/30 cycles at 0°	Mains power quality should be that of a typical commercial or hospital environment.
Voltage Interruptions on power supply input lines IEC 61000-4-11	0% $U_T$ , 250/300 cycle	0% $U_T$ , 250/300 cycle	
Radiated RF IEC 61000-4-3	10 V/m	10 V/m	Performed over 80 MHz to 2.7 GHz

**Note 1:** At 80 MHz and 800 MHz, the higher frequency range applies.

**Note 2:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Electromagnetic Immunity

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ME Equipment is used exceeds the applicable RF compliance level above, the ME Equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the ME Equipment.

<sup>1</sup> U<sub>i</sub>: Rated voltage for the equipment.

## Test Specifications for ENCLOSURE PORT IMMUNITY to RF Wireless Communication Equipment

Test Frequency (MHz)	Band (a) (MHz)	Service (a)	Modulation (b)	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
385	380-395	TETRA 400	Pulse modulation (b) 18 Hz	1,8	0,3	27
450	430-470	GMRS 460, FRS 460	FM (c) +/- 5 kHz deviation 1 kHz sine	2	0,3	28
710	704-787	LTE Band 13, 17	Pulse modulation (b) 217 Hz	0,2	0,3	9
745						
780						
810	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation (b) 18 Hz	2	0,3	28
870						
930						
1 720	1 700-1	GSM 1800; CDMA	Pulse	2	0,3	28

Test Frequency (MHz)	Band (a) (MHz)	Service (a)	Modulation (b)	Maximum Power (W)	Distance (m)	Immunity Test Level (V/m)
1 845	990	1900; GSM 1900; DECT; LTE Band 1, 3, 4. 35: UMTS	modulation (b) 217 Hz			
1 970						
2 450	2 400-2 570	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation (b) 217 Hz	2	0,3	28
5 240	5 100-5 800	WLAN 802.11 a/n	Pulse modulation (b) 217 Hz	0,2	0,3	9
5 500						
5 785						

**Note:** If necessary to achieve the IMMUNITY TEST LEVEL, the distance between the transmitting antenna and the ME EQUIPMENT or ME SYSTEM may be reduced to 1 m. The 1 m test distance is permitted by IEC 61000-4-3.

**(a)** For some services, only the uplink frequencies are included.

**(b)** The carrier shall be modulated use a 50% duty cycle square wave signal.

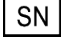
**(c)** As an alternative to FM modulation, 50% pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

## Recommended Separation Distances

<b>Recommended Separation Distance Between Portable and Mobile RF Communication Equipment and the ME Equipment</b>	
<p>The ME Equipment is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the ME Equipment can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the ME Equipment as recommended below, according to the maximum output power of the communication equipment.</p>	
<b>Rated maximum output power of transmitter (W)</b>	<b>Separation Distance According to Frequency of Transmitter (m)</b>
	$d = 0.6 * \text{Sqrt}(P)$
0.01	0.06
0.1	0.19
1	0.6
10	1.9
100	6
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance <math>d</math> in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p>	
<p><b>Note 1:</b> At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p><b>Note 2:</b> These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>	

# Symbols

The following symbols may appear on the product or product labeling:

Symbol	Description	Symbol	Description
	Follow instructions for use		Consult instructions for use
	Mark of conformity to European medical device directive 93/42/EEC		Separate collection for electrical and electronic equipment (WEEE)
IP22	Protection from solid foreign objects $\geq 12.5$ mm diameter and against ingress from vertically falling water drops when enclosure tilted up to $15^\circ$	IP47	Protection from solid foreign objects $< 1$ mm diameter and against immersion between 15 centimeters and 1 meter in depth
	Non-Sterile		Federal Communications Commission (FCC) Licensing
	Authorized representative in the European community		Recyclable
IC Model:	Innovation, Science and Economic Development Canada (ISED)	FCC ID:	Identifies unit has been registered as a radio device
	Caution		Product contains no PVC (polyvinyl chloride) material
	Manufacturer		Not made with natural rubber latex
	Date of manufacture YYYY-MM-DD		Catalog number (model number)
	Storage temperature range		Masimo reference number
	Keep dry		Serial number

Symbol	Description	Symbol	Description
	Storage humidity limitation		Do not use if package is damaged
	Atmospheric pressure limitation		AC current
	Wireless Symbol level		The names and content of the toxic and hazardous substances or elements shall be provided in the product instruction manual
	China Restriction of Hazardous Substances	--	--
	Instructions/Directions for Use/Manuals are available in electronic format @ <a href="http://www.Masimo.com/TechDocs">http://www.Masimo.com/TechDocs</a> <b>Note:</b> eFU is not available in all countries.		

## Citations

[1] The Masimo SET Technology has been validated for no motion accuracy in human blood studies on healthy adult male and female volunteers with light to dark pigmented skin in induced hypoxia studies in the range of 70%-100% SpO<sub>2</sub> against a laboratory co-oximeter.

[2] The Masimo SET Technology has been validated for motion accuracy in human blood studies on healthy adult male and female volunteers with light to dark pigmented skin in induced hypoxia studies while performing rubbing and tapping motions, at 2 to 4 Hz at an amplitude of 1 to 2 cm and a non-repetitive motion between 1 to 5 Hz at an amplitude of 2 to 3 cm in induced hypoxia studies in the range of 70%-100% SpO<sub>2</sub> against a laboratory co-oximeter.

[3] The Masimo SET Technology has been validated for low perfusion accuracy in bench top testing against a Biotek Index 2 simulator and Masimo's simulator with signal strengths of greater than 0.02% and transmission of greater than 5% for saturations ranging from 70% to 100%.

[4] The Masimo SET Technology has been validated for pulse rate accuracy for the range of 25-240 bpm in bench top testing against a Biotek Index 2 simulator and Masimo's simulator with signal strengths of greater than 0.02% and transmission of greater than 5% for saturations ranging from 70% to 100%.

\*Registered trademark of Fluke Biomedical Corporation, Everett, Washington.

# Service and Maintenance

## Cleaning

The Bedside Station and Masimo Chip are reusable devices. The devices are supplied and used non-sterile.

The Masimo Chip should be cleaned before and after it has been applied to a user and/or in accordance with local and governmental regulations to minimize the risk of cross-contamination.

## Smart Phone Cleaning

To properly clean the smart phone, refer to the smart phone's Operator's Manual or Directions For Use.

## Bedside Station Cleaning

**CAUTION:** Check the Bedside Station for possible cracks or opening before cleaning.

**CAUTION:** Do not allow liquids to enter the interior of the Bedside Station.

The outer surfaces can be cleaned either with a soft cloth dampened with a mild detergent and warm water solution or they can be wiped down with the following cleaning solutions:

- 70% isopropyl alcohol
- 1:10 bleach to water solution (0.5% sodium hypochlorite)
- Super Sani-Cloth® Wipes (55% isopropyl alcohol, 0.5% quaternary ammonium chloride)
- Windex® (1.5% 2-Butoxyethanol, 1.5% ethylene glycol hexyl ether, 5% isopropyl alcohol)
- Formula 409® Antibacterial All-Purpose Cleaner (1.5% Lauramine oxide, 0.4% n-Alkyl dimethyl benzyl ammonium chloride)

## Masimo Chip Cleaning

**WARNING:** Before cleaning, make sure the sensor and chip are not applied to the hand.

To surface clean the chip and wireless receiver:

1. Wipe all surfaces of the chip and wireless receiver with one of the following:
  - a. 70% Isopropyl alcohol
  - b. 10% (1:10) chlorine bleach to water solution
  - c. Quaternary ammonium chloride solution
2. Inspect for visible debris and repeat the above cleaning step as needed.
3. Dry cleaned parts before use.

**CAUTIONS:**

- To avoid permanent damage to the chip and wireless receiver, do not use undiluted bleach (5% - 5.25% sodium hypochlorite) or any other cleaning solution not recommended.
- Do not immerse the chip and wireless receiver in any liquid solution.
- Do not sterilize by irradiation, steam, autoclave or ethylene oxide.

## Customer Support

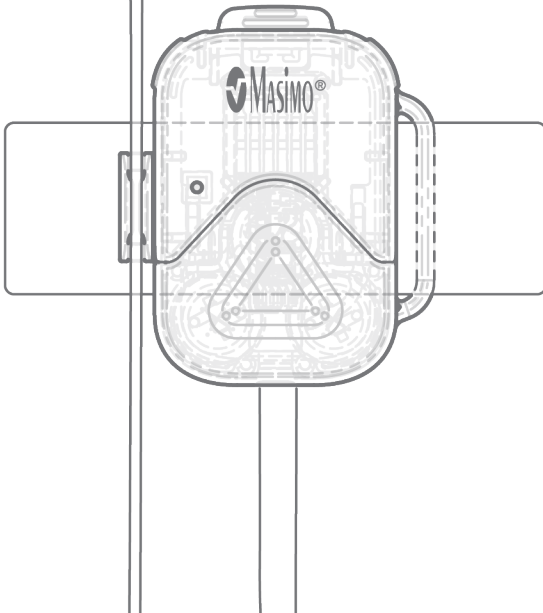
For answers to frequently asked questions (FAQ's) and product support, along with troubleshooting for your Masimo SafetyNet Alert product, please go to [www.mymasimo.co.uk/safetynet-alert](http://www.mymasimo.co.uk/safetynet-alert) or email [customercare@masimo.com](mailto:customercare@masimo.com).

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