

# Radius T<sup>°</sup> Setup Instructions

**Warning:** Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

## Download & View Instructions for Use

Before using this device, follow the instructions for use in the Support section of [masimo.com/radius-t](https://masimo.com/radius-t). Call Masimo Customer Service to request printed Instructions or if product support is required. Contact information can be found on the website.



Apple and the Apple Logo are registered trademarks of Apple Inc. App Store is a registered service mark of Apple. Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.



## GET TO KNOW YOUR SYSTEM

### Masimo Radius T° App


The app collects and displays temperature data and trends from the paired wearable sensor. You can also set high temperature notifications.



### Radius T° Sensor\*

Worn on your chest for up to 8 days, the sensor measures body temperature and sends your data to the Masimo Radius T° App.

\*Sensor can be used for people ages 5 and up.

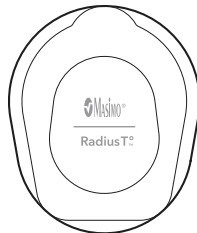


## GETTING STARTED

### Set up App




Download the Masimo Radius T° App and follow instructions for getting started.


### Pair & Apply Sensor

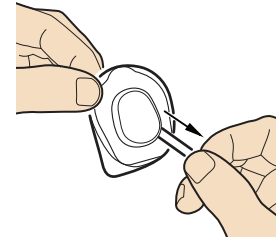

1


Remove sensor from pouch and pull tab to activate battery.

Pair the sensor with your phone: Under app Settings , select sensor user's name and Connect Sensor, then follow on-screen instructions.

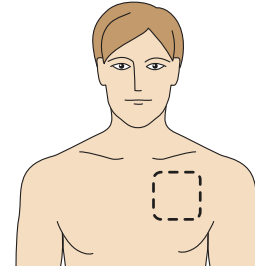


2

Choose an application site on the upper chest below the left collar bone. Make sure the site is hair-free, clean and dry.

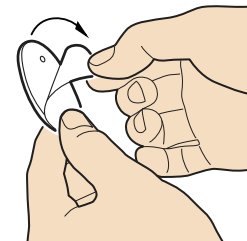

3

Remove plastic film from the back of the sensor.

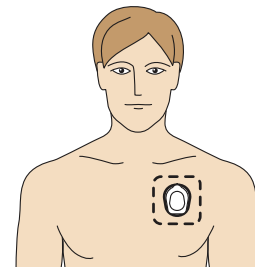

4

Apply sensor to the site and press around the adhesive to ensure it is secure.

For additional resources, including videos and instructions, visit [masimo.com/radius-t](https://masimo.com/radius-t)

# APP OVERVIEW Note: App screens may change over time

Current person being viewed

App settings

Temperature trend direction (Arrow indicates change of  $\geq 0.5$  °F vs. 30 minutes ago.)

Temperature range:  
Green = Normal  
Yellow = Fever  
Red = High fever

Latest temperature

Add notes & reminder

Temperature trend & notes

Note:  
A temperature reading may take up to 15 minutes to appear in the app after you first apply the sensor.


Time period you want to view

Highest & lowest temperatures

Time in normal, fever & high fever range

Hourly high/low temperatures (tap to see details)

## TIPS FOR USING APP

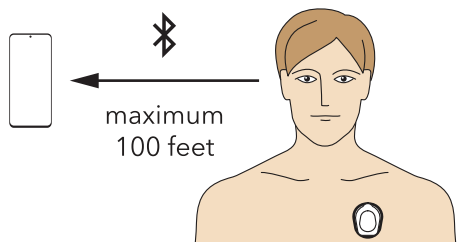

Before pairing, make sure you've enabled Bluetooth on your phone. The sensor must be within Bluetooth range of your phone to receive data.




You can leave the app whenever you want, but do not close the app down or your temperature session will disconnect.


## CARING FOR YOUR SENSOR



If battery light is orange or red, this means that the battery is close to depletion.

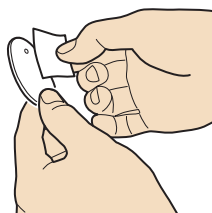

If needed, to remove and reapply the sensor, gently wipe the exposed adhesive with an alcohol swab and allow to dry before reapplying.

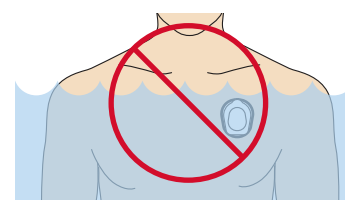

Sensor can be worn in the shower but should not be submerged.